

Initial A™: Single use distal fibula sterile kits

Initial A™ - Fibula & Syndesmosis kits content

Indications:


The implants of the Initial A™ range are intended for the fixation of fractures, osteotomies and pseudarthroses of the distal and the diaphyseal fibula, the distal tibia and for the syndesmotic repair in adults.

Contraindications:


- Serious vascular deterioration, bone devitalization.
- Pregnancy.
- Acute or chronic local or systemic infections.
- Lack of musculo-cutaneous cover, severe vascular deficiency affecting the concerned area.
- Insufficient bone quality preventing a good fixation of the implants into the bone.
- Muscular deficit, neurological deficiency or behavioral disorders, which could submit the implant to abnormal mechanical strains.
- Allergy to one of the materials used or sensitivity to foreign bodies.
- Serious problems of non-compliance, mental or neurological disorders, failure to follow post-operative care recommendations.
- Unstable physical and/or mental condition.



> Initial A™ - Fibula kit

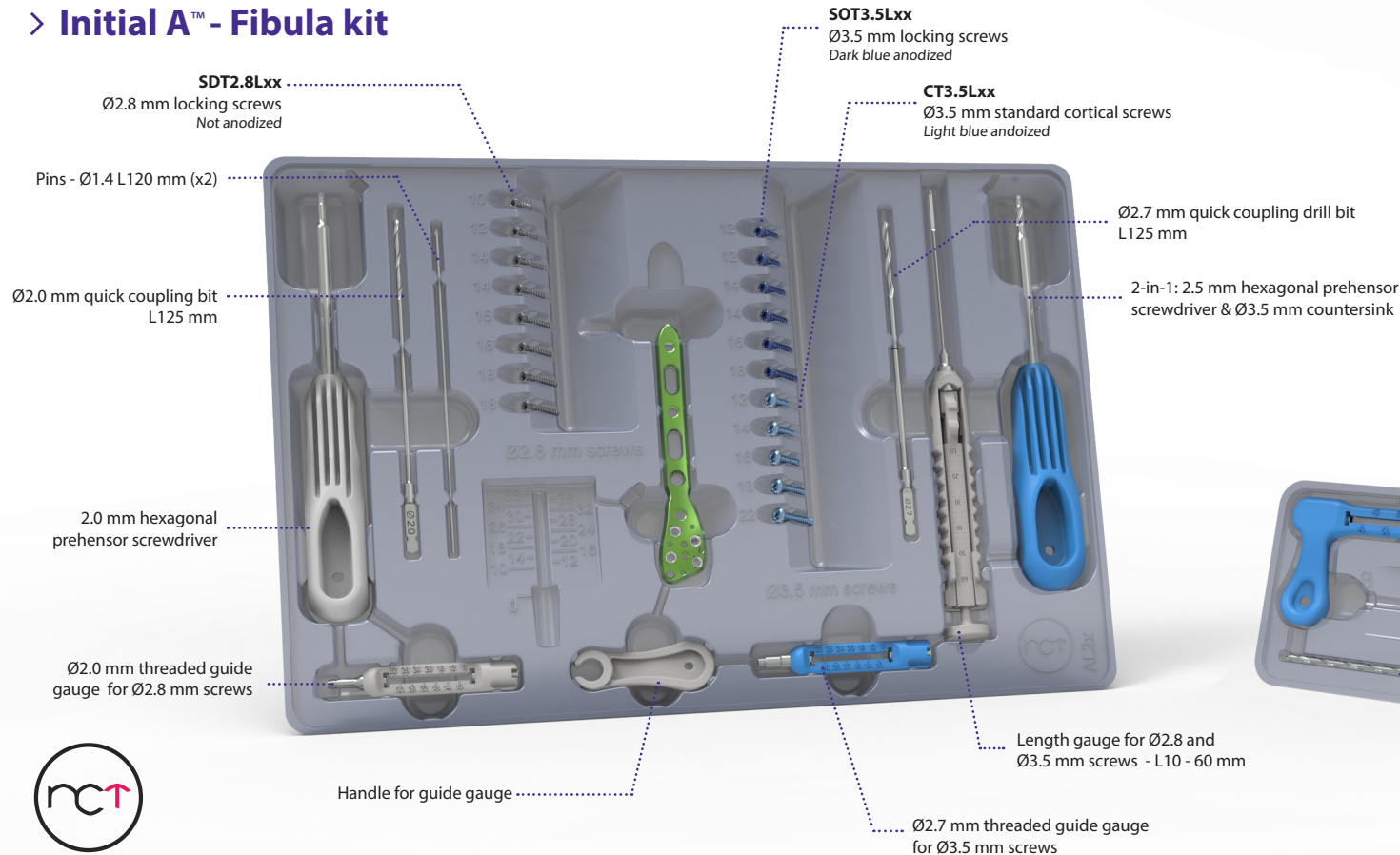


> Initial A™ - Syndesmosis kit

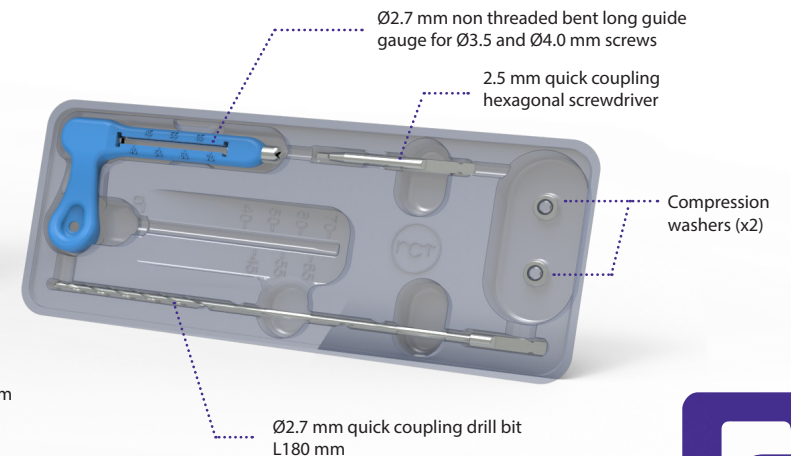

NEWCLIP
TECHNICS




Initial A™ - Fibula kits references: Implants

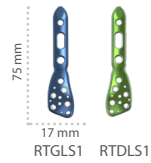

Precontoured lateral plates
with polyaxial locking fixation
Standard - Size 1

KIT-AL1G or KIT-AL1D* Distal fibula kit - Standard - Left or right - Size 1		
Ref.	Description	Qty
RTxLS1	Lateral plate for distal fibula - Standard - Size 1 (left or right)	1
SDT2.8L10	Locking screw - Ø2.8 mm - L 10 mm	1
SDT2.8L12	Locking screw - Ø2.8 mm - L 12 mm	1
SDT2.8L14	Locking screw - Ø2.8 mm - L 14 mm	2
SDT2.8L16	Locking screw - Ø2.8 mm - L 16 mm	2
SDT2.8L18	Locking screw - Ø2.8 mm - L 18 mm	2
SOT3.5L12	Locking screw - Ø3.5 mm - L 12 mm	1
SOT3.5L14	Locking screw - Ø3.5 mm - L 14 mm	1
SOT3.5L16	Locking screw - Ø3.5 mm - L 16 mm	1
SOT3.5L18	Locking screw - Ø3.5 mm - L 18 mm	1
CT3.5L14	Standard cortical screw - Ø3.5 mm - L 14 mm	1
CT3.5L16	Standard cortical screw - Ø3.5 mm - L 16 mm	1
CT3.5L20	Standard cortical screw - Ø3.5 mm - L 20 mm	1
CT3.5L24	Standard cortical screw - Ø3.5 mm - L 24 mm	1

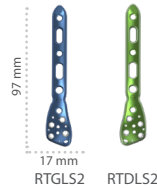

Precontoured lateral plates
with polyaxial locking fixation
Standard - Size 2

KIT-AL2G or KIT-AL2D* Distal fibula kit - Standard - Left or right - Size 2		
Ref.	Description	Qty
RTxLS2	Lateral plate for distal fibula - Standard - Size 2 (left or right)	1
SDT2.8L10	Locking screw - Ø2.8 mm - L 10 mm	1
SDT2.8L12	Locking screw - Ø2.8 mm - L 12 mm	1
SDT2.8L14	Locking screw - Ø2.8 mm - L 14 mm	2
SDT2.8L16	Locking screw - Ø2.8 mm - L 16 mm	2
SDT2.8L18	Locking screw - Ø2.8 mm - L 18 mm	2
SOT3.5L12	Locking screw - Ø3.5 mm - L 12 mm	2
SOT3.5L14	Locking screw - Ø3.5 mm - L 14 mm	2
SOT3.5L16	Locking screw - Ø3.5 mm - L 16 mm	1
SOT3.5L18	Locking screw - Ø3.5 mm - L 18 mm	1
CT3.5L12	Standard cortical screw - Ø3.5 mm - L 12 mm	1
CT3.5L14	Standard cortical screw - Ø3.5 mm - L 14 mm	1
CT3.5L16	Standard cortical screw - Ø3.5 mm - L 16 mm	1
CT3.5L18	Standard cortical screw - Ø3.5 mm - L 18 mm	1
CT3.5L22	Standard cortical screw - Ø3.5 mm - L 22 mm	1




Precontoured lateral plates
with polyaxial locking fixation
Standard - Size 3

KIT-AL3G or KIT-AL3D* Distal fibula kit - Standard - Left or right - Size 3		
Ref.	Description	Qty
RTxLS3	Lateral plate for distal fibula - Standard - Size 3 (left or right)	1
SDT2.8L10	Locking screw - Ø2.8 mm - L 10 mm	1
SDT2.8L12	Locking screw - Ø2.8 mm - L 12 mm	1
SDT2.8L14	Locking screw - Ø2.8 mm - L 14 mm	2
SDT2.8L16	Locking screw - Ø2.8 mm - L 16 mm	2
SDT2.8L18	Locking screw - Ø2.8 mm - L 18 mm	2
SOT3.5L12	Locking screw - Ø3.5 mm - L 12 mm	3
SOT3.5L14	Locking screw - Ø3.5 mm - L 14 mm	2
SOT3.5L16	Locking screw - Ø3.5 mm - L 16 mm	1
SOT3.5L18	Locking screw - Ø3.5 mm - L 18 mm	1
CT3.5L12	Standard cortical screw - Ø3.5 mm - L 12 mm	1
CT3.5L14	Standard cortical screw - Ø3.5 mm - L 14 mm	2
CT3.5L16	Standard cortical screw - Ø3.5 mm - L 16 mm	1
CT3.5L18	Standard cortical screw - Ø3.5 mm - L 18 mm	1
CT3.5L22	Standard cortical screw - Ø3.5 mm - L 22 mm	1



Precontoured lateral plate
with polyaxial locking fixation
Narrow - Size 1

KIT-AL1S Distal fibula kit - Narrow symmetrical - Size 1		
Ref.	Description	Qty
RTSLN1	Lateral plate for distal fibula - Narrow symmetrical - Size 1	1
SDT2.8L14	Locking screw - Ø2.8 mm - L 14 mm	1
SDT2.8L16	Locking screw - Ø2.8 mm - L 16 mm	2
SDT2.8L18	Locking screw - Ø2.8 mm - L 18 mm	1
SOT3.5L12	Locking screw - Ø3.5 mm - L 12 mm	2
SOT3.5L14	Locking screw - Ø3.5 mm - L 14 mm	2
SOT3.5L16	Locking screw - Ø3.5 mm - L 16 mm	1
CT3.5L14	Standard cortical screw - Ø3.5 mm - L 14 mm	1
CT3.5L16	Standard cortical screw - Ø3.5 mm - L 16 mm	1
CT3.5L22	Standard cortical screw - Ø3.5 mm - L 22 mm	1

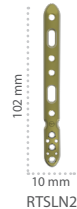

Precontoured lateral plate
with polyaxial locking fixation
Narrow - Size 2

KIT-AL2S Distal fibula kit - Narrow symmetrical - Size 2		
Ref.	Description	Qty
RTSLN2	Lateral plate for distal fibula - Narrow symmetrical - Size 2	1
SDT2.8L14	Locking screw - Ø2.8 mm - L 14 mm	1
SDT2.8L16	Locking screw - Ø2.8 mm - L 16 mm	2
SDT2.8L18	Locking screw - Ø2.8 mm - L 18 mm	1
SOT3.5L12	Locking screw - Ø3.5 mm - L 12 mm	2
SOT3.5L14	Locking screw - Ø3.5 mm - L 14 mm	2
SOT3.5L16	Locking screw - Ø3.5 mm - L 16 mm	2
CT3.5L12	Standard cortical screw - Ø3.5 mm - L 12mm	1
CT3.5L14	Standard cortical screw - Ø3.5 mm - L 14 mm	1
CT3.5L16	Standard cortical screw - Ø3.5 mm - L 16 mm	1
CT3.5L22	Standard cortical screw - Ø3.5 mm - L 22 mm	1

* Each kit includes one plate (left or right)
NB : Additional sterile screws can be ordered separately .

Initial A™ - Syndesmosis Kit references

INITIAL A™ - SYNDESMOSIS KIT

Ref.	Description
KIT-AMS	Syndesmosis kit

ASSOCIATED STERILE SCREWS - AVAILABLE SEPARATELY

Ref.	Description	Qty
CT3.5L40-ST	Standard cortical screw - Ø3.5 mm - L 40 mm - STERILE	2
CT3.5L45-ST	Standard cortical screw - Ø3.5 mm - L 45 mm - STERILE	3
CT3.5L50-ST	Standard cortical screw - Ø3.5 mm - L 50 mm - STERILE	3
CT3.5L55-ST	Standard cortical screw - Ø3.5 mm - L 55 mm - STERILE	3
CT3.5L60-ST	Standard cortical screw - Ø3.5 mm - L 60 mm - STERILE	2
CT3.5L65-ST	Standard cortical screw - Ø3.5 mm - L 65 mm - STERILE	2
QT4.0L40-ST	Lag screw - Ø4.0 mm - L 40 mm - STERILE	3
QT4.0L45-ST	Lag screw - Ø4.0 mm - L 45 mm - STERILE	3
QT4.0L50-ST	Lag screw - Ø4.0 mm - L 50 mm - STERILE	3
QT4.0L55-ST	Lag screw - Ø4.0 mm - L 55 mm - STERILE	3
QT4.0L60-ST	Lag screw - Ø4.0 mm - L 60 mm - STERILE	3
QT4.0L65-ST	Lag screw - Ø4.0 mm - L 65 mm - STERILE	3



NEWCLIP TECHNICS
PA de la Lande Saint Martin, 45 rue des Garotières
44115 Haute Goulaine, France
+33 (0)2 28 21 23 25
orders@newcliptechnics.com
www.newcliptechnics.com

NEWCLIP USA
642 Larkfield Center
Santa Rosa CA 95403, USA
+1 707 230 5078
customerservice@newclipusa.com
www.newclipusa.com

NEWCLIP GmbH
Pröllstraße 11
D-86157 Augsburg, Deutschland
+49 (0)821 650 749 40
info@newclipgmbh.com
www.newclipgmbh.de

NEWCLIP Australia
3B/11 Donkin Street
West End 4101, Australia
+61 (0)2 81 886 110
sales@newclipaustralia.com
www.newcliptechnics.com

NEWCLIP TECHNICS Japan K.K.
KKK Bldg 502, 3-18-1 Asakusabashi
Taisho-Ku, Tokyo, 111-0053, Japan
+81 (0)3 58 25 49 81
Fax: +81 (0)3 58 25 49 86
www.newcliptechnics.fr

NEWCLIP TECHNICS IBERIA
Calle Frederic Mompou, 4b
Sant Just Desvern, 08960 Barcelona, España
+34 938 299 526
contact@newclipiberia.com
www.newcliptechnics.com